




Instructions for Use: UnderGuard for Plasma™

Brand Name of Product	UnderGuard for Plasma™
Generic Name of Product	Tray liners
Product Code Number(s)	MAT001, MAT002, MAT003, MT-2030, MT-1520, MT-1224.
Purpose of Product	For use under the tray to protect outer wrap from sharp corners and imperfections of sterilization trays.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> Manufactured from spun polypropylene. The unique purple color helps distinguish this protective layer from sterilization wrap.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	15°C-30°C and 30-70% RH
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	To protect outer wrap from sharp corners and imperfections of sterilization trays.
Preparation for Use	<ol style="list-style-type: none"> Select a liner which will fit best the target basket. If using with wrapped tray, liner must be inside the wrap and may be used inside or under the basket. If using with a closed container, liner must be used inside the container, either inside or below the basket.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> Place liner either inside or beneath basket. In the event you are using a container, the liner should be placed in case first. Place items in tray. Use proper sterility assurance indicators and follow hospital protocol. Wrap or close package. Place in gas-plasma/VHP or steam sterilizer.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	MATxxx are number 5 for recycling.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	

Packaging	
Sterilization	Material compatible with gas-plasma/VHP and steam sterilization up to 137°C
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-07-05 msmith