

ASSEMBLY INSTRUCTIONS



figure 1



figure 2



figure 3



figure 4

CAUTION

INSPECT CONTENTS IMMEDIATELY AND FILE CLAIM WITH DELIVERING CARRIER FOR ANY DAMAGE. SAVE YOUR BOX AND ALL PACKING MATERIALS. YOU ARE RESPONSIBLE FOR DAMAGE TO YOUR UNIT IF RETURNED IMPROPERLY PACKED.

Parts List

- (2) Chrome posts*
- (1) Utility cart handle*
- (1) Pack split sleeves(per shelf)**
- (2) Chrome shelves*
- (4) Bumper donuts
- (4) Stem casters

*Quantity depends on model ordered

**4 per pack

1 Install first shelf

Referencing numbers on posts, snap plastic tapered split sleeves onto each post at desired location of bottom shelf. The thicker part of the taper goes toward the bottom make sure rib on split sleeve is seated into a groove on the post as shown in figure 1.

Note: The number covered by the split sleeve indicates approximate height of shelf. Holding the first shelf on it's side, insert the posts through the corner collars until split sleeves are seated snugly with collars as shown in figure 2. Stand the unit upright.

2 Install additional shelves

Snap split sleeves onto post at the location for the second shelf. Lower shelf onto the post until the split sleeves are snugly seated within the corner collars. Repeat for each additional shelf working from the bottom to the top

Note: All additional shelves must be mounted before securing any shelves.

3 Secure shelves

Place a board across the corners and tap with a hammer or mallet as shown in figure 3.

Caution: Do not hammer directly on wire or corner.

4 Install casters

Place the unit on its side. Place donut bumpers onto bottom end of post. Bottom is end with 1½ inches ungrooved. Insert stem of caster into bottom end of post. Drive in caster using a hammer or mallet, several solid blows are usually required as shown in figure 4.